
CLEAR

FEATURES & ADVANTAGES

The new dimension for teeth straightening



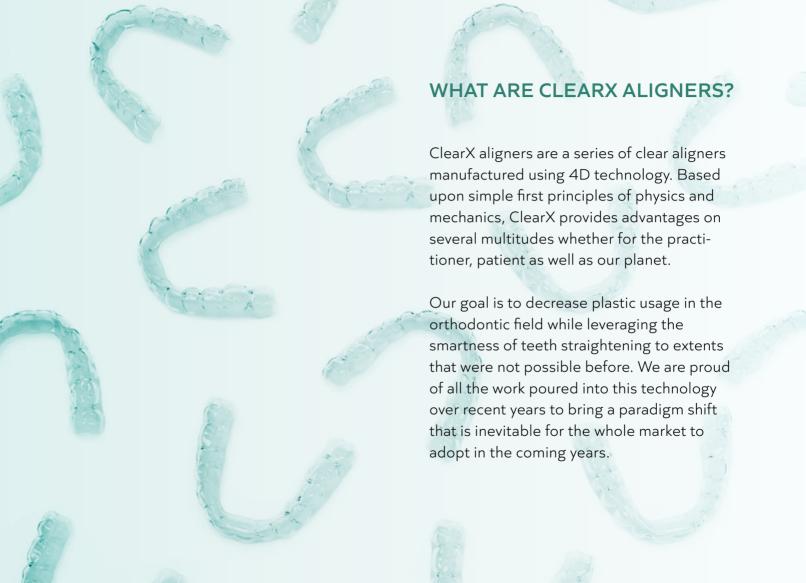

HOW TO START AND BOOST YOUR ALIGNER



STEP 1 DOWNLOAD THE APP

Scan the QR Code or search for **ClearX** in Google play store or Apple store to download the app.



ClearX app



INSTRUCTIONS video

STEP 2 IS IT YOUR BOOSTING DAY?

Connect the booster with ClearX app.

Make sure to turn on bluetooth, location and notifications.







Pour water in the booster, then preheat the booster.



STEP 3 BOOST THE ALIGNER

Put the aligner in the booster using the tweezer.



Activate the boosting.




CLEARX MOBILE APP

For a seemless and well controlled treatment flow, the aligner booster is accessed easily and individually per patient.

The software guides the patient through the steps of aligner activation.



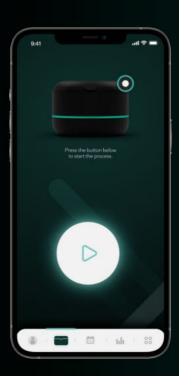





CLASSICAL TO 4D TECH

System comparison for 12 months case treatment

K CLEAR OTHER TRADITIONAL METHODS

24
3d models
(per arch)

24
Aligners
(per arch)



Movement set-ups



Treatment time



CLEARX 4D TECHNOLOGY

13
3d models
(per arch)



12
Aligners



Movement set-ups



Treatment time



ADVANTAGES

of ClearX aligners over traditional aligner system



ACCURACY

Each aligner acts as a retainer in the first week of wearing



TIME EFFICIENT

Teeth movement acceleration is higher



LESS PAINFUL

on aligner insertion



LESS PLASTIC

Less aligner and 3D model count



BE ECO-FRIENDLY

You can return the booster and the aligners at the end of your treatment in order to help us to reduce electronic and plastic waste for a pollution free environment.






CLEARX RESULTS

4.5 months treatment


The 4D technology used in the making of ClearX has superior results in comparison to the classical aligner treatments. Thanks to the recurrent aligner patented method, the teeth movement lag, often seen in classical methods, is overcome and continuously being adjusted on monthly basis.

While the treatment accuracy is mandatory, we didn't forget how crucial speed is. ClearX provides 35% shorter treatment durations in comparison to other classical systems in the market, with ClearX up to 1mm/month tooth movements could be expected.

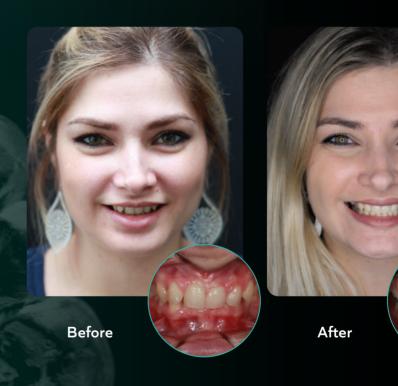

SMART ALIGNERS

BOOSTER

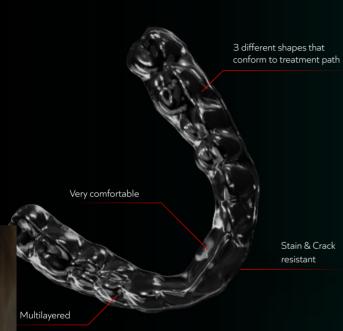



1 week



Patient receives an aligner that has shape memory for 3 different shapes 1 week



Activate aligner the first time with the booster

1 week



Activate aligner the second time with the booster




Go to next aligner and repeat the process

IN-LAB

THERMOMECHANICAL

TESTING

Mechanical tests under flexure | Results

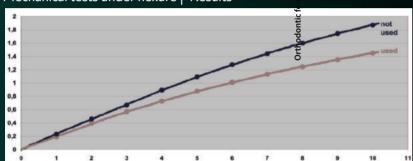





Cooperation

We have set cooperations to study thermomechanical behaviour, shape memory characteristics, and orthodontic forces exhibited by the material.







WHAT DOCTORS SAY



Dr. S. KandilGermany - K Line Europe

"The future of orthodontics is great and we are determined to play a major role in it. 4D ortho is a fundamental move for the whole industry. It opens the door to so many new possibilities. Accurate, fast, eco-friendly and smart are all concievable meanings 4D brings to orthodontics."



Dr. G. Singh

"The only thing permanent in life is change. And aligner technology per say was disruptive for orthodontics. Now ClearX is likely to be disruptive for the aligner industry. Looking forward to change with 4D."



Dr. Charlston UY
Phillipines

"4D aligner technology is a game changer. It will revolutionize the field of Aligner Orthodontics and I'm glad that K Line will lead that revolution."



Dr. R. Hughes

United Kingdom

"The new K Line ClearX 4D technology brings fantastic innovation and disruption to the clear aligner industry. ClearX is a millennialisation of orthodontics, a very exciting concept for both clinicians and patients."



Dr. Arozo Ah.

Germany

"I worked with different aligner systems and nothing convinced me more than the 4D aligners from K Line. Even complex orthodontic cases are possible to treat in a simple and efficient way. My patients were surprised by the very fast and pleasing results. It's definitely my number one choice in teeth straightening."



Dr. C. Dewdney

United Kingdom

"K Line is an already established, dynamically growing and ever innovative orthodontic company that has focused, very uniquely, cutting edge scientific research and development with ecological awareness and responsibility to reduce our "plastic footprint". The utilisation of smart materials, coupled with patient booster units to track aligner modifications and instigate them is revolutionary!"



Any questions? We're here to help!

Contact us +49 172 6898684 info@clearxaligners.com





K LINE EUROPE GMBH

SEGRO PARK DÜSSELDORF SÜD Unit 203 K Bonner Straße 203-393 40589 Düsseldorf, GERMANY www.clearxaligners.com